



# USER MANUAL




SLD Care - Toilet Lift Seat



SLD Care technology-based company integrates product technology development, design, manufacture, and sales, involving aged care, aged support, and rehabilitation. Shengledi is committed to home-based rehabilitation. The design and development of this kind of product, pioneering and innovative, help the elderly and knee osteoarthritis patients live independently through products.

1. DIMENSIONS

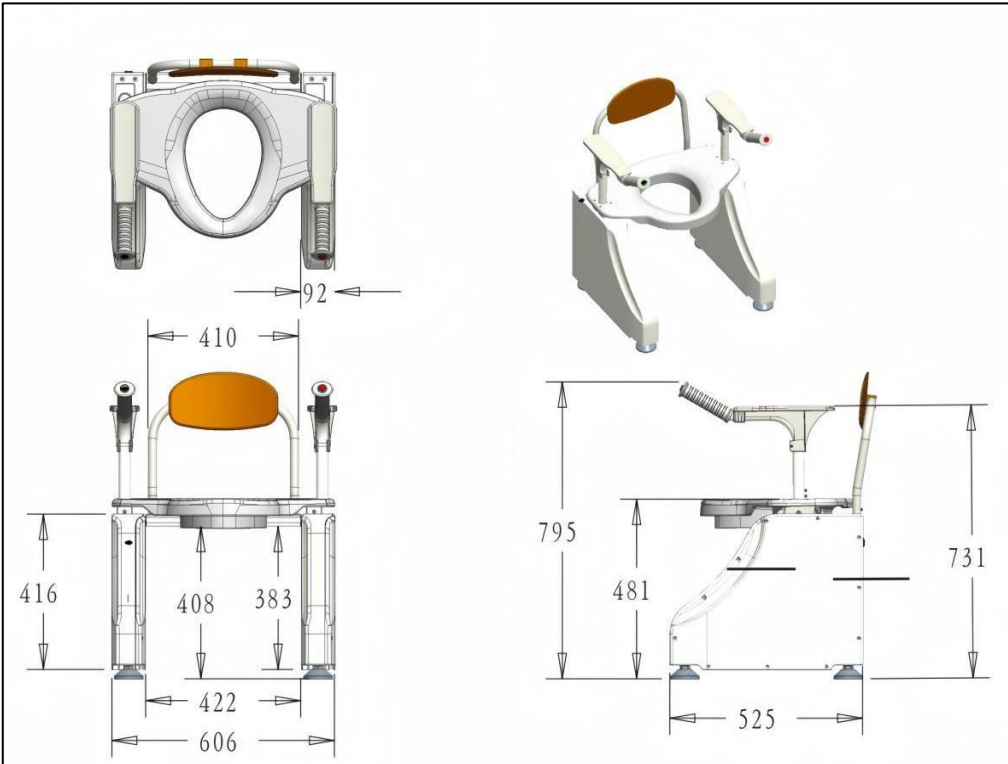


2. FUNCTIONAL ACCESSORIES



FUNCTIONAL ACCESSORIES	
1	Armrest
2	Switch button
3	Lithium battery
4	Electric quantity display
5	Remote control
6	Alarm button
7	Backrest
8	Bedpan

### 3. ACCESSORIES PACKAGE

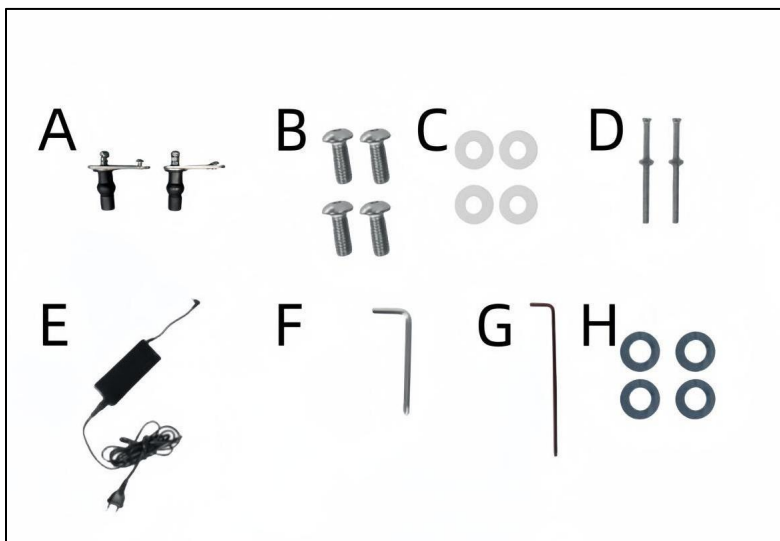

ACCESSORIES PACKAGE	
A	Fixing bolt fastening screw
B	Small screws
C	Washers
D	Large screws
E	Charger
F	Allen wrench + screwdriver
G	Allen wrench round head
H	Metal washers

## 4. HANDLES

Connect the right handle (switch button) with the cable located in the central frame. Check that the white dots on both wires match. Then insert the left handle into the other hole of the central frame.



## 5. BACKREST


Tighten the handles with the small screws . Place a plastic washer and a metal washer (H) between the tube and the screw, and use the Allen wrench (G) to tighten it well.



## 6. ARMRESTS

Position the armrest securely onto the handles, ensuring a firm press until a satisfactory engagement sound is heard, indicating proper attachment.

For enhanced safety, it is imperative to securely screw the armrests in place. To accomplish this task, utilize the provided large screws (D), which are accompanied by a small nut. With the assistance of the Allen wrench and screwdriver (F), firmly tighten the screws to ensure stability and durability.

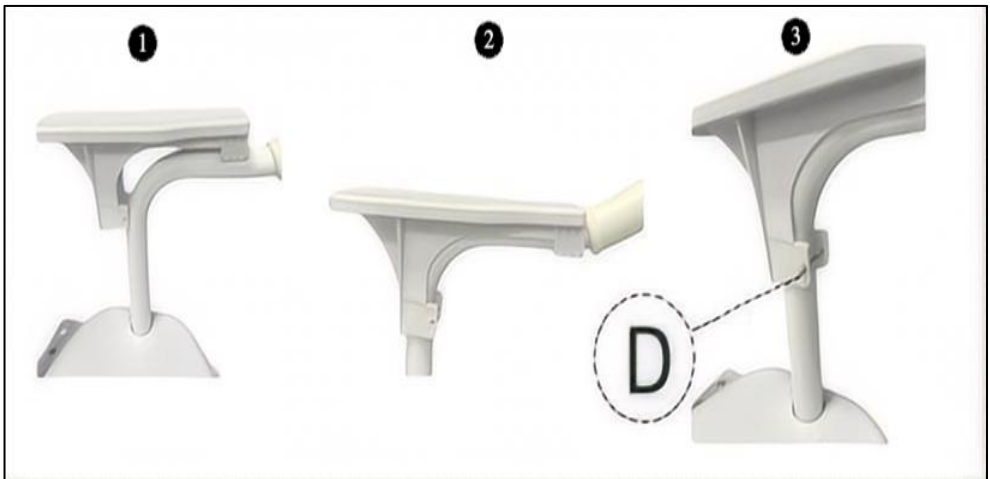



## 7. LEG ADJUSTMENT



The legs have been adjusted to their minimum height. To carry out the adjustment process:

1. Rotate the legs in a counterclockwise direction.
2. Upon attaining the preferred height, secure the nut by rotating it until it is firmly situated at the apex of the leg.
3. Verify that it is firmly and securely fastened in place.



## 8. INTSTALL THE LIFTER

To install the lifter Frame on your toilet, unscrew the lid and toilet bowl (toilet lids), leaving only the ceramic base. Slide the structure towards the toilet, leaving the lower part of the inner hole of the structure inside the toilet.

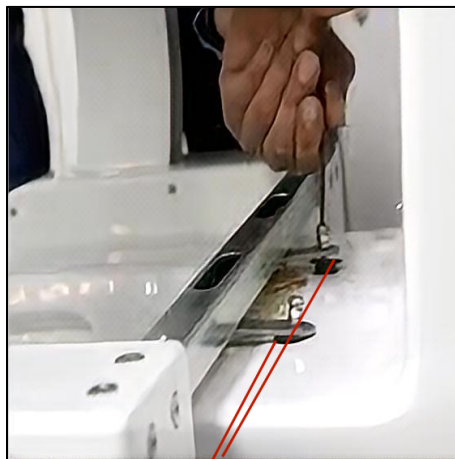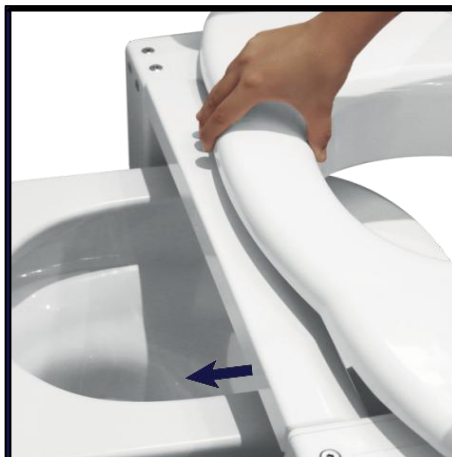

## 9. BATTERY

Finally, connect the charger to the frame and the plug to a light socket.

The red light on the frame indicates that the cable is properly connected and the frame is ready to use.



## 10. CONTROL BUTTON

Press and hold the left button to raise the frame and assist the user to sit on the toilet. Press the right button continuously to lower the frame and help the user to sit on the toilet. The Toilet frame tilts to an angle of 33°.

If you choose the luxury model, you can directly use the remote control to control up and down.



## 11. TECHNICAL SPECIFICATIONS

- Maximum loading: 200kg/kg
- Power voltage: 220V
- battery output nominal voltage: Direct current 24V
- Working voltage: 24V
- Product rated power: 130w
- Water-proof grade: IP44

## WARNINGS

- Read the instructions carefully before installation.
- Users with limited physical capabilities must be supervised or assisted when using the device.
- Do not exceed the maximum weight supported by the device, which is 330 lb.
- The handles are intended to serve as a support for standing and/or sitting, using both at the same time. Do not drop your entire body weight on only one of the handles.
- Do not stand on this product. It is not a climbing aid.
- Do not apply 100% of the weight only on one armrest, use only half of the body weight on each of the armrests. In case of overloading, the armrest may break.
- Check that all components are properly assembled and screwed together before use.
- Keep away from heat sources.

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

## CLEANING AND MAINTENANCE

- Clean regularly by hand with a damp cloth. Do not use abrasive cleaning agents or rinse with water.

- Disinfect the product by wiping all accessible surfaces with disinfectant  
(after disassembly, if possible). All cleaning products used must be effective, compatible with each other and protect the materials to be cleaned.
- If the structure is to be used by several people, it must be disinfected at each use.
- For disinfection in hospital environments, follow the instructions of a hygiene specialist.

## WARRANTY

SLD guarantees that this device is free from defects. The warranty shall remain in force for 1 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear requiring periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than SLD or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- SLD is not liable for accidental damage or damage to property.