



## Installation procedure



1. Take out the product and all accessories, install the hydraulic pump cover, screw the small screws on the cover.



2. Install the foot lever on the hydraulic pump and tighten the short hex socket screw with a wrench.



3. Close the side



4. If the hole at the upper end of the solid shaft of the liquid pump rod is not in the middle position, press the foot rod to the very bottom and do not release it. At the same time, rotate the solid shaft to align the hole with the small screw on the decorative cover. At the same time, press the solid shaft to the very bottom and then release the foot rod.



5. Release the brakes on all four wheels of the chassis lid.



6. Install the hydraulic pump on the bottom rack, insert 4 M8\*30 screws into the gasket, and screw in from the bottom. (Don't tighten the screws yet)



7. Insert the two thick round tubes into the mounting holes of the base frame, and screw in the black twist screws to press the round tubes tightly.



8. Put the armrests on both sides of the circular tube.



9. Direct the hydraulic pump to the middle hole of the armrest frame, and insert the M6\*40 screws, screw in the small nuts, and then use a wrench to tighten the screws.



10. Lay the shifter on its side and tighten the fixing screws under the base frame with a wrench.



11. Insert the left and right mounts into the support rods respectively, then insert the screws of the M8\*50 and tighten them with the wrench screws.



12. Close the switch handle and put the screw cap on the well cover.



13. Install the armrests on both sides of the seat frame with M5\*35 screws and tighten the screws with a wrench.



14. Install the armrests on both sides of the seat frame with M5\*35 screws and tighten the screws with a wrench.



15. Load the toilet bucket into the shelf.



16. Screw the two support posts into the serving table.



17. Insert the table into the plastic holes on both sides of the shifter.