

Walking Aid YC8230 Instruction Manual

Warm Tips: Please read the instructions carefully before use.



Dear User:

- Thank you for purchasing the Yade walking aid YC8230.
- Before using the multifunctional walking aid, please carefully read the user manual and correctly use and understand it.
- Please keep this manual safe after reading it for easy reference and reference at any time.

Product Description

- A walking aid is a device that can be used to support the elderly, the sick, and other people who have lost the ability to walk and walk on their own, as well as to go out for a walk like a normal person.
- YC8230 walking aid is an upgraded version of Yade's aluminum alloy walking aid with an armrest, which combines many features of walking aids and adds some new functions for users under the careful research of our designers and visits to old customers. Its main structure is made of aluminum alloy, which makes the walking aid lighter and not easy to rust. The walking aid has front wheels and sled feet so that the user can walk easily without too much force, which is more humanized.
- The walking aid adopts a one-key interchangeable interactive/fixed mode, which is more convenient and quicker than the traditional small C-card interchangeable mode.
- YC8230 walking aid is suitable for patients with lower limb dysfunction, poor muscle strength of the lower limbs, or the elderly to assist standing and walking, also suitable for post-surgery rehabilitation training walking.

Product Brand: Yade	Safe Load-capacity: 100kg
Product Size (L*W*Armrest Height Above Ground): 52*60*(106-121)cm	
Gross Weight: 5kg	Net Weight: 4.15kg

Warm Tips:

1. Before use, please check whether the product is damaged or not solid, all the marble buttons should be popped out of the hole before use.
2. The walking aid with wheels is strictly prohibited from being forced on the uneven, bumpy, sloping road surface, otherwise it may cause irreparable trauma to the product as well as injury to the user (recommended for indoor use).
3. Please check the product regularly to see if it is in good condition.

- Soft Grip
- Fold Button
- Widened and Thickened Front Rod
- Soft Armrest
- PVC
- PVC Grip

- Aluminum Alloy Main Frame
- Interchangeable Interactive/fixed Switch Button
- Height Adjustable Leg
- Aluminum Alloy Reinforced Side Rod
- 5-inch Front Wheel
- Sled Footrest



How to Use/Install

1. Open the packaging and remove the product.




2. Hold the PVC handles on both sides with both hands, then pull them outward. When you hear a "snap" sound, the walking aid will fully open.



3. Install the front wheel leg (with the wheel on the outside).



4. Press the marble and install the armrest assembly.

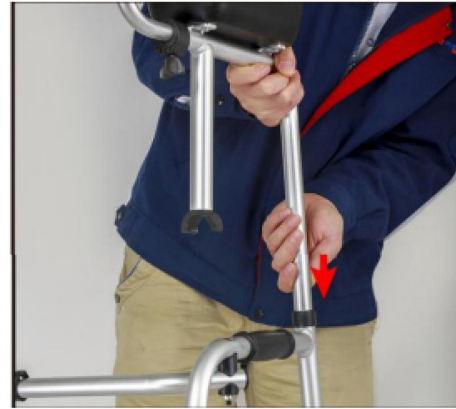

5. The marbles should be exposed out of the holes and pressed down a bit to make the snaps snap onto the aluminum tube. please pull the snap upward a little bit first, and when the snap is loosened, then press the
(On the contrary, if you need to remove the arm rest assembly, marble at the back, and then lift the arm rest assembly upward).



6. Loosen the twist screw first, press the marble, adjust the grip to the right distance, and finally tighten the twist screw.



Use Demonstration


Please do not shift the center of gravity.



Please do not use to assist in getting up.



Please do not push backward.

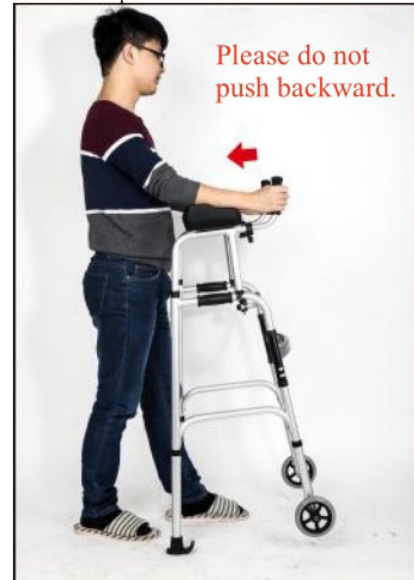

Fixed
Interactive

Fixed and Interactive Methods:

1. Press the folding button first, and retract the walking aid telescopic tube a little bit inward.
2. Press the switching button, and adjust the marble to the desired position (the marbles in the upper frame are fixed, and the marbles in the lower frame are interactive).