

# TOILET SEAT MANUAL



This toilet seat with armrest is convenient for the user to go up and down the toilet. Please note that the user cannot support the whole body weight with armrest alone, otherwise it may damage the product or cause damage.

## Specification

Distance between armrest	51.4cm
Overall height	37cm
Height of armrest	23.9cm
Overall length	47cm
Raised range( 3 level)	5cm, 10 cm, 15cm

## Safety

1. Do not lean out of the toilet seat, so as not to fall down from the seat, this toilet seat is only for indoor use;
2. Please check carefully whether all accessories are tightened before use.
3. Check the weight limit and weight load before use, do not use overweight.
4. Do not change the product.

# Installation

1. Please check the product for no damage before use.
2. Detach toilet cover.
3. Insert the armrest into the toilet seat hole.



4. Unscrew the black buttons on both sides of the toilet seat and place on the toilet.



5. Tighten the black buttons on both sides until they are not loose and firmly locked on the toilet.



6. Flip-up armrest

